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
Read the instruction manual before use and keep it for future reference.

Important safety instructions

0803123
COMFORT SLING WITH HEAD
SUPPORT (MESH L)

EN INSTRUCTION FOR USE

Size colour marking

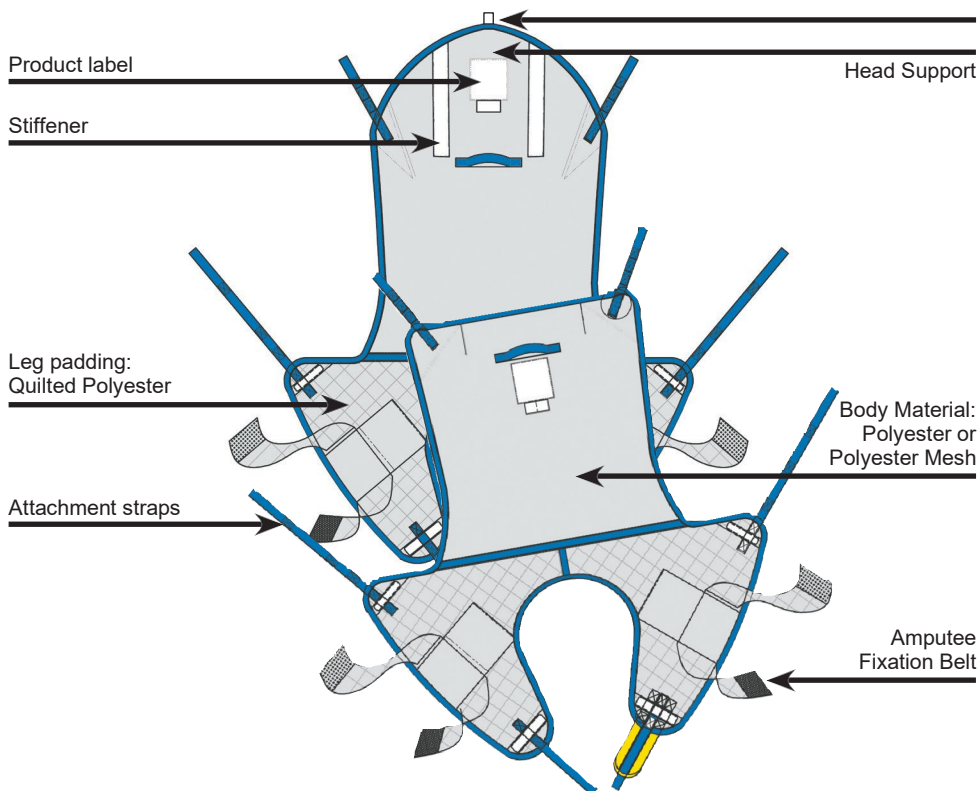Washing instructions:

Warning!

Always read this instruction and the instruction for the lifter to be used with the sling

Technical Specifications

Code	Description	Max. Weight	Material
0803123	COMFORT SLING WITH HEAD SUPPORT (MESH L)	275kg	Polyester

Definition in this IFU

Warning: Means Safety warning. Failure to understand or obey this warning may result in injury to you or to others.
Note: Means important information for the correct use of this system or equipment.

Intended Use

The Comfort Sling with Head Support is designed with split legs and full shaped body for users who requires head support. Suitable for most transfers in a seating or laying position and where user have little neck and/or head control.
The Comfort Sling will assist the caregiver for lifting and transfer a patient in either a sitting or laying position, from and to the bed and/or wheelchair.

Safety instructions

- Always check the label is readable and that the sling is in good condition without signs of wear and tear.
- Always make sure not to exceed the Max. Weight defined for the product.
- Always read the instruction manual for the lifting equipment to be used with the sling.

Before and after every use

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the sling, read this instruction manual and the instruction manual for the lifting equipment to be used
3. When soiled or stained the sling must be washed before use.
4. **Warning:** Check the sling for any fraying, loose stitching, attachments or any deteriorations.
5. After use, check sling according to the step 3 and 4 above and store in a dry area.

Service & Maintenance

The sling shall be checked by authorized personnel every 6 months according to ISO10535

Customer support/Trouble shooting

- Please refer to the manual for the lifting equipment used
- In doubt please contact your dealer who supplied the product

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

Expected service life & Disposal

Expected Service Life varies between 1–5 years, depending on usage and number of washing cycles. When washed once a week the sling has an Expected Service Life of 2 years. Expected Service Life is based upon a correct usage and washings procedure in accordance with this IFU. Dispose the slings according to the local regulations.

Storing and Shelf life

The sling should be stored in a dry and clean environment. The shelf life is 3 years in original packaging. Store in temperatures between 0°-40° C (+32°-104°F)


Product care & Maintenance

- Always follow the instructions on the product label and check the sling for deteriorations.
- Mark the sling with the specific patient name and keep in dry and clean environment.

Applying the Comfort Sling in Chair

1. Lean the user slightly forward and position the Sling behind the users back and head. The label side facing outward and the center line of the sling aligned with the user's spine. Carefully push the sling as far down the patient lower back as possible. (See picture 1)


Note! If used on a leg amputated user, attach fixation belt on the amputee side and secure it around the residual limb.

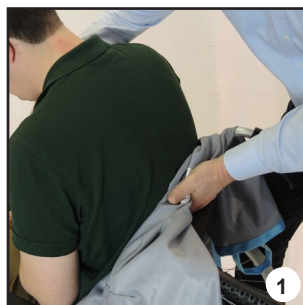

Place the sling legs alongside the user's legs. Fold the sling legs and carefully feed it underneath the user's legs. (See picture 2,3,4)

After positioning the sling legs cross the short yellow strap and put both thigh straps through it. Then cross each thigh strap with it's own leg strap. (See picture 5,6)

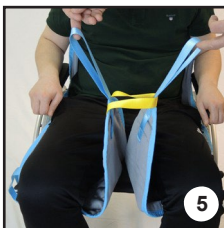
Note! If required, two flat slide sheets can be used to apply the sling. (See picture 7,8)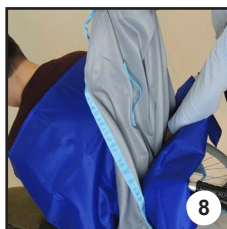
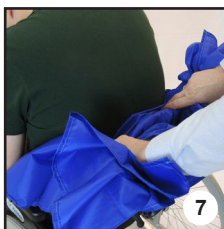


2. Recline the user into the sling, ensuring the arms are contained within the sling.

Warning! Make sure that the sling won't get caught in the armrest or anywhere else. (See picture 9,10)

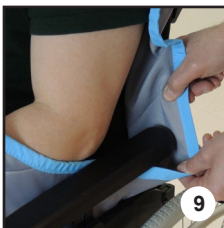
3. Position the Hanger bar carefully directly above the user. (See picture 11,12)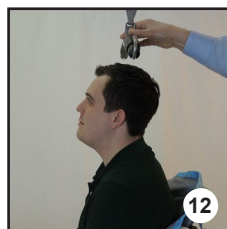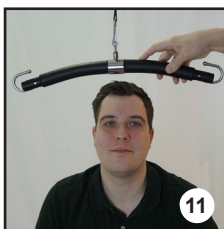

4. Place the four loop straps on the hanger bar attachments hooks and pull the strap down until locked in position. (See picture 13,14)

Note: Always check the lifter instruction manual details.

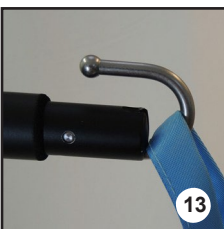5. Check user position and carefully make the required adjustments to achieve the best comfort for the user.

Check the support of the head and that arms and legs are in the right positions.
(See picture 15)

6. Raise the hanger bar until all straps start to tighten and become tense enabling you to check **ALL** the straps are securely attached.
(See picture 16)

7. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort.
(See picture 17)


8. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose.
(See picture 18)


Release the loop straps or clips from the hanger bar by pulling/pushing the strap upwards.
(See picture 19,20)

Note: Always check the lifter instruction manual details.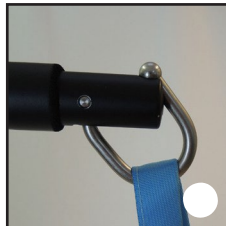

19

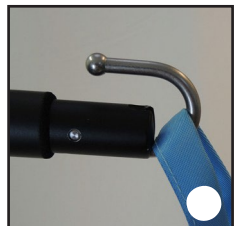20

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

Applying the Comfort Sling in Laying position



1. Roll the user to the side, facing away from you.

Roll up top half or the sling and gently tuck it under the user.

(See picture 1)


Fit the sling by folding it on the middle. The label side facing outwards. Then place the sling lengthwise with the users back. (See picture 2,3,4)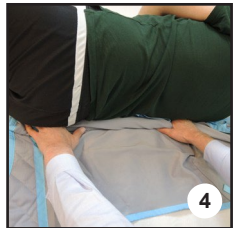
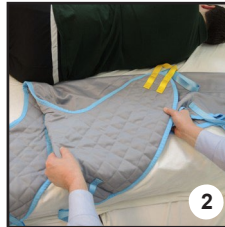


2. Roll user to the other side, now facing you. Gently pull the top half of the sling through, when fully unraveled flatten it. Roll the user to a backside position. (See picture 5,6)


Note! If used on a leg amputated user, attach fixation belt on the amputee side and secure it around the residual limb.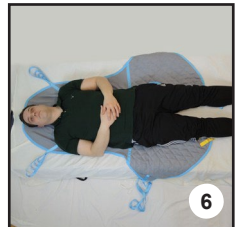


3. Insert sling legs under the user thigh. (See picture 7,8)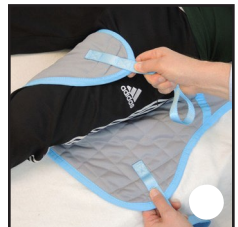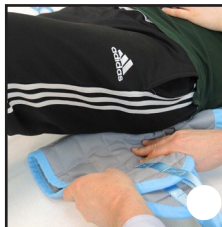

After positioning the sling legs cross the short yellow strap and put both thigh straps through it.
Then cross each thigh strap with its own leg strap.
(See picture 9,10,11)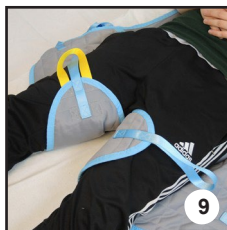

Note! If required, two flat slide sheets can be used to apply the sling.
(See picture 12,13)


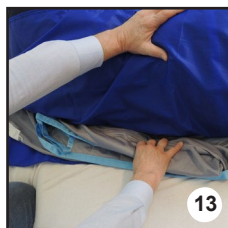
4. Position the hanger bar carefully above the user.
(See picture 14,15)

5. Place the four loops straps on the hanger bar attachment hooks and pull the straps down until locked in position.
(See picture 16,17)

Note: Always check the lifter instruction manual details.

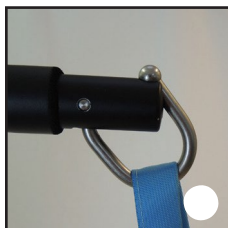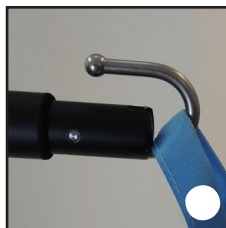6. Check user position and carefully make the required adjustments to achieve the best comfort for the user. Check the support of the head and that arms and legs are in the right positions.
(See picture 18)

Note: If required, recline the bed so the user is in a half-sitting position.

7. Raise the hanger bar until all straps start to tighten and become tense, enabling you to check that **ALL** the straps are securely attached."

8. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort. Remember to support the user's head when raising from the bed.
(See picture 19,20)



9. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose.
(See picture 21)

10. Release the loop strap from the hanger bar by pulling the strap upwards.
(See picture 22,23)

Note: Always check the lifter instruction manual details.

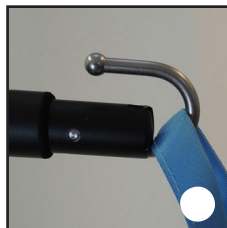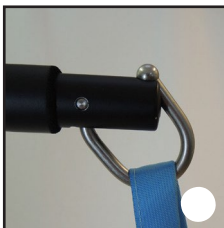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