REF MASTER-TIM PLUS

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BRUGSANVISNING

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USER MANUAL

Tilt-in-space shower and commode transport chair 3in1

Product code: MASTER-TIM PLUS

1. Characteristics

Shower and commode transport chair is made of powder coated aluminum frame. It features flip-up armrests for easy entry and exit for people with reduced mobility and seat & backrest angle adjustment. It is equipped with a seat with wedge made of PU, headrest, four swivel 5" wheels with brakes, tiltable footrests and a sanitary container. The construction of the chair makes it possible to slide over the toilet seat.

The set includes: frame, seat, headrest, headrest tube, two headrest locking knobs, four swivel 5" wheels with brakes, four C-type wheel mounting buckles, backrest with armrests, two armrest extensions, two backrest locking knobs, footrests with footplate, sanitary container with lid.

Elements of the chair

- A. Headrest
- B. Headrest locking knob
- C. Handle
- D. Headrest tube
- E. Armrest
- F. Backrest
- G. Armrest extension
- H. Backrest locking knob
- I. Seat
- J. Footrest latch
- K. Footrest adjustment
- L. Wheel with brake
- M. Footrest
- N. Sanitary container

2. Application

Shower and commode transport chairs are designed for people who have problems with unassisted movement and for patients with dysfunctions and reduced mobility. The chair is suitable for indoor use only. The chair may only be used with the assistance of an accompanying person. The chair functions as a

shower and commode transport chair owing to the sanitary container located under the seat and four swivel castors. It facilitates everyday hygiene activities. The chair is especially recommended to people who have problems with mobility which are caused by: paralysis, obesity or abnormal underweight, degenerations or deformity of limbs, degenerations or joints injuries, limbs injuries or amputations, heart and blood vessel disorder and circulatory disorders.

3. Assembly and use

Figure 1. Fix the seat on the frame of the chair by pressing it until it locks into place (the front of the seat is indicated by the wedge). Mount the backrest with armrests in the holes at the back of the frame. Align the holes and lock them with two backrest locking knobs. Insert the sanitary container into the slides from the back of the seat.

Figure 2. Install the four wheels into the main frame. Align the holes at the desired height and lock each wheel with a C-type buckle. The chair has a 4-step height adjustment. **Note!** All wheels must be set at the same height.

Figure 3. Insert the footrests into the main frame from the side, and then rotate it to the front until the footrest latch locks in place. The footrests have a 3-step height adjustment - adjustable by the locking knobs M6.

Figure 4. Fix both armrest extensions to the armrest mounting holes, then lock in place by snapping the securing pins.

Figure 5. Fix the headrest tube in the mounting hole at the top of the backrest, adjust it in the desired position, then lock it with the locking knob.

Figure 6. Fix the headrest in the mounting hole of the headrest tube, adjust it to the desired position, and then lock it with the locking knob.

The product is ready for use.

CHAIR HANDLING

- Check the condition of the brakes, construction and castors. Check the proper assembly of the components and their adjustment.
- Before using, make sure that the chair's width matches the user, as the chair is not equipped with width adjustment.
- The chair is designed to be used for short distances and inside only. The chair doesn't have a shock absorber, so long-term use is not recommended.

Note! Do not use the chair if any defects or deficiencies are noticed.

FLIP-UP FUNCTION OF ARMRESTS

- Lift the armrest upwards with a gentle movement.
- **Do not** use the armrest as a support while standing up. If help is required – ask for assistance.
- Armrest extensions can be fixed/removed at any time to make the wheelchair easier to use.

BRAKES

To lock the castors, press the red pedals with your foot. To unlock the wheel, press the black levers inside the red pedals with your foot.

FOOTRESTS

- The footplates should be in vertical position when getting in or out of the chair. **Do not use the footplates as a support while getting in or out of the chair due to the risk of falling over.**

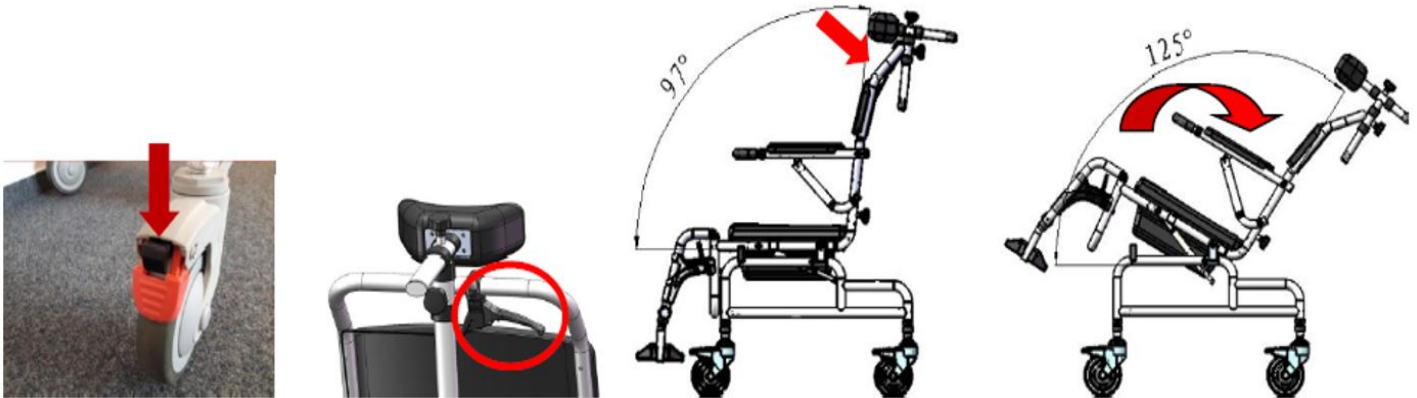
GETTING ONTO THE CHAIR

- It is essential to lock the brakes while getting in or out of the chair, changing the user's position or getting off.
- While changing place, the chair should be located as close as possible to the place of transfer. The chair should touch the bed or other seat being used. If the user cannot change his/her seat independently, transfer the user to the chair.

- The user should sit as deep as possible in the middle of the seat in order to avoid chair instability.
- Armrest extensions can be fixed/removed at any time to make it easier to get on or off the chair.

ANGLE ADJUSTMENT OF SEAT & BACKREST

- Before adjusting the angle, it is essential to lock the brakes.
- Press and hold the adjustment handle, then set the desired angle of the seat & backrest. After setting the desired angle, release the adjustment handle.

4. Technical data

Overall width	54 cm
Inner width	46 cm
Overall depth	106 cm
Seat height from the ground	55-61 cm
Overall height	115-136 cm
Seat dimensions	44 x 44 cm
Weight	13 kg
Maximum capacity	150 kg
Angle adjustment of seat & backrest	max. 30°

5. Notes

Timago International Group does not bear responsibility for improper use of the chair and failure to comply with safety regulations.

6. Cleaning and maintenance

Carry out the maintenance of the chair components regularly:

- The seat can be cleaned with a damp cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge.
- Plastic elements should be cleaned with damp cloth with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.
- Empty and disinfect the toilet bucket and toilet seat after each use. Use cleaning products according to producer's recommendations.

7. Storage and transport conditions

The product should not be exposed to direct sunlight.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that, for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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