

INSTRUCTIONS FOR USE

Aluminium adjustable cane


Product code: ELEGANT



1. Characteristics

Aluminium cane has a profiled handle and a wide range of adjustment thanks to its stepped length adjustment. The shaft is finished with a soft rubber cap.

- a. handgrip
- b. wrist strap
- c. height adjustment – locking pin
- d. rubber tip

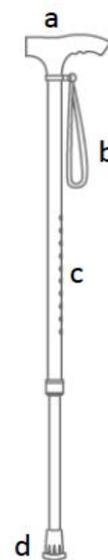

2. Application

The adjustable cane is ancillary equipment which is used to increase the area of support in case of patients with reduced mobility. The cane is recommended to patients who need help and support while walking and it can also be used as an aid in keeping balance after the injuries of lower limbs or trunk.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs which may hinder the user to support on the cane.

3. Adjustment

To adjust the cane, press the metal pin and select appropriate position of the inner tube. Lock the position of the tubes relative to each other with the use of metal pin. The cane is adjustable in 2.5 cm increments within the range of 70-93 cm.

- To set the proper length of the cane, stand upright and bent upper limb in elbow joint at the angle of 20°.
- The cane should be adjusted individually to the user so that the handgrip is at the hip joint level.
- Before use, ensure that the cane is stable and the locking pin fully protrudes through adjustment hole of the cane.

NOTE! In case of doubts concerning the technique of moving with the cane, consult a person who has appropriate knowledge or qualifications.

4. Technical data

TYPE: adjustable

Colours: silver/blue, brown

Height adjustment: 70-93 cm

Maximum capacity: 136 kg

Cane weight: 0.38 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the cane, failure to comply safety regulations and misuse.

6. Maintenance and cleaning

- Clean the cane using only water and mild detergents.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- It is recommended to check the rubber tip regularly. Replace worn tip immediately.

7. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

8. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

The entire range is available to you from our distributors.
Find out more at www.timago.com
Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Poland

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



02/2022(II)