

INSTRUCTIONS FOR USE
Elastic ankle support
Product code: TGO-C OSS 612



1. Characteristics

Elastic ankle support is made of seamless orthopaedic compression fabric. Anatomical shape is properly fitted to the ankle joint, affects the direction of compression and precise stabilization. Modern and spatial fabric is air permeable and removes excessive heat and moisture limiting the development of bacterial flora.

2. Application

Ankle support is used to stabilize the joint and soothe pain which occurs in physical strains i.a. in sport.

Indications for use:

- Pain syndrome of joints caused by strains and micro-injuries
- Minor twists and strains of ligaments
- Chronic instability

Contraindications:

Do not use in case of:

- Skin abrasion
- Eczema
- Wound and other skin injuries
- After application of warming gel or ointment
- Individual contraindications which were determined by the doctor during interview and physical examination

3. Notes

Timago International Group does not bear responsibility of improper maintenance of the product, failure to comply safety regulations and misuse.

4. Fitting instruction

- Pull the elastic support on from the bottom to the ankle joint.

Note! Only properly matched size guarantees reduction of pain.

Note! Long-term use may result in reduction of muscle tone and dysfunction of the limb.

5. Technical data

SIZE	CIRCUMFERENCE OF CALF (CM)
S	21-23
M	23-25
L	25-27
XL	27-29

6. Maintenance

- Hand wash in warm water. Do not tumble, chlorinate, iron, bleach, dry clean, use tumble dryer.
- Dry at room temperature far from intensive heat source.

Note! Maintenance information can be found on the material label.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
---	------------------	---	--------------

	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021