

INSTRUCTIONS FOR USE

Padded elbow strap

Product code: TGO-C SL 903



1. Characteristics

Padded elbow strap consists of a strap fastened by Velcro and a silicone pad which affects attachments of the muscles with a controlled pressure. The insets of the silicone pad perform a micro-massage while movements of elbow joint reducing the pain. The strap reduces the risk of overloads or tears of muscle attachments of extensor and flexor of the forearm and hand.

2. Application

Elbow strap is used in medical treatment in order to relief the area of condyles of humerus.

Indications for use:

- Tears and strains of proximal attachments of the forearm muscles
- Acute and chronic pain syndrome
- Non-invasive treatment
- Preventive treatment of sport injuries

Contraindications:

Do not use in case of:

- Skin abrasion
- Eczema
- Wound and other skin injuries
- After application of warming gel or ointment
- Individual contraindications which were determined by the doctor during interview and physical examination

3. Notes

Timago International Group does not bear responsibility of improper maintenance of the product, failure to comply safety regulations and misuse.

4. Fitting instruction

- Put the strape on with silicone pad placed in the area of lateral epicondyle of humerus. Wrap the strap below elbow joint and fasten with Velcro.
- When the strap is too long, cut its end using scissors to the appropriate length so that it can be attached to the strape.

Note! Only properly matched size reduces the risk of overload and strain of muscle attachments of extensor and flexor of the forearm and hand. Long-term use may result in reduction of muscle tone and dysfunction of the limb.

5. Technical data

SIZE	UNIVERSAL
------	-----------

6. Maintenance

- Hand wash in warm water. Do not tumble, chlorinate, iron, bleach, dry clean, use tumble dryer.
- Ensure the Velcro is properly fastened in order not to damage the material.
- Dry at room temperature far from intensive heat source.

Note! Maintenance information can be found on the material label.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the

product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021