

# INSTRUCTIONS FOR USE

## Knee brace with patella stabilizer

Product code: TGO-C SR 501



### 1. Characteristics

Knee brace with patella stabilizer is made of elastic material which possesses perfect properties in terms of air permeability and removal of moisture. There is an opening with foamed ring in the central part of the brace which positions the brace and relieves the patella. The brace is additionally equipped with lateral spiral stays ensuring firm and stable movements of flexion and extension of the knee joint. Anatomic shape ensures snug fit of the brace.

### 2. Application

Knee brace may be used in preventive treatment and in post-injury rehabilitation in order to limit the range of mobility in the knee joint and to improve functional activities. The brace relieves the patella, stabilizes the knee and leads the flexion and extension movements limiting lateral movements in unstable joint.

#### Indications for use:

- Pathological changes of the patella
- Rheumatoid inflammations of the knee joint
- Patellar apex syndrome
- Dislocation of the patella
- Minor failure of ligaments
- Preventive treatment of excessive strain and injuries
- Rheumatism

#### Contraindications:

Do not use in case of:

- Skin abrasion

- Eczema
- Wound and other skin injuries
- After application of warming gel or ointment
- Individual contraindications which were determined by the doctor during interview and physical examination

### 3. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

### 4. Fitting instruction

- Pull the elastic support on from the bottom to the knee joint so that the opening with foamed ring is placed on the central part of the patella and lateral stays are located in the axis of the joint from lateral and medial sides.
- Fasten upper strap with Velcro.
- Using scissors cut the end of Velcro to the appropriate length so that it can be easily attached to the orthosis.

**Note!** Only properly matched size guarantees correct stabilization of knee joint and relief of the patella.

**Note!** Long-term use may result in reduction of muscle tone and dysfunction of lower limb.

### 5. Technical data

SIZE	CIRCUMFERENCE OF PATELLA (cm)
S	34-36
M	36-38
L	38-41
XL	41-44
XXL	44-47

## 6. Maintenance

- Hand wash in warm water. Do not tumble, chlorinate, iron, bleach, dry cleaning, use tumble dryer.
- Ensure the Velcro is fastened in order not to destroy the material.
- Dry at room temperature far from intensive heat source.

**Note!** Maintenance information can be found on the material label.

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.  
Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!



Timago International Group  
Spółka z o.o. i Spółka – Spółka komandytowa  
ul. Karpacka 24/12  
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00  
F.: +48 33 499 50 11  
E.: [info@timago.com](mailto:info@timago.com)



Rev. II, date 20.05.2021