REF

NEXO

Timago
International Group

- PL INSTRUKCJA OBSŁUGI**
- CS NÁVOD K OBSLUZE**
- DA BRUGSANVISNING**
- DE BENUTZERHANDBUCH**
- EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ**
- EN USER MANUAL**
- ET KASUTUSJUHEND**
- HR KORSNIČKE UPUTE**
- HU FELHASZNÁLÓI ÚTMUTATÓ**
- IT MANUALE D'USO**
- LT NAUDOJIMO INSTRUKCIJA**
- LV LIETOTĀJA INSTRUKCIJA**
- PT MANUAL DO UTILIZADOR**
- RO MANUAL DE INSTRUCȚIUNI**
- SK NÁVOD NA POUŽITIE**
- SL NAVODILA ZA UPORABO**
- SV BRUKSANVISNING**

PL INSTRUKCJA OBSŁUGI	4
CS NÁVOD K OBSLUZE	7
DA BRUGSANVISNING	10
DE BENUTZERHANDBUCH	13
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	17
EN USER MANUAL	20
ET KASUTUSJUHEND	23
HR KORSNIČKE UPUTE	26
HU FELHASZNÁLÓI ÚTMUTATÓ	29
IT MANUALE D'USO	32
LT NAUDOJIMO INSTRUKCIJA	35
LV LIETOTĀJA INSTRUKCIJA	38
PT MANUAL DO UTILIZADOR	41
RO MANUAL DE INSTRUCȚIUNI	44
SK NÁVOD NA POUŽITIE	47
SL NAVODILA ZA UPORABO	50
SV BRUKSANVISNING	53

1. Wykonanie

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1. Characteristics

The shower and commode chair is made of durable, water resistant material. It has height-adjustable aluminum legs finished with non-slip rubber tips. The backrest and seat can be additionally equipped with a soft, padded cushions. Flip-up armrests greatly increase stability and helps patient get up from the chair. The chair comes with a removable plastic sanitary container with a lid.

Set includes: Chair frame, seat and backrest cushion, sanitary container with lid.

Elements of the commode chair:

- A. Padded backrest
- B. Flip-up armrest
- C. Seat with padded cushion and wedge
- D. Sanitary container
- E. Front frame
- F. Height adjustment
- G. Anti-slip rubber tip
- H. Hinge

2. Application

The shower and commode chair is an auxiliary equipment that facilitates the user to perform hygienic activities. It is recommended for people who have problems with independent movement, with dysfunctions and movement limitations, which prevents or hinders independent performance of activities of daily life. The chair has a toilet function thanks to the provision of a sanitary receptacle under the seat. It facilitates carrying out the user's daily physiological and care activities. The chair is also recommended for use in showers and baths, in home and hospital settings.

3. Assembly, adjustment and usage

UNFOLDING

Step 1: Take out chair from the box.

Step 2: Grab chair by the back of the seat (C) and backrest (A), slowly unfolding until the chair hinges (H) are fully unfolded.

Step 3. Adjust the height of the legs with the locking pins.

Step 4: Take out the backrest and seat cushions from the box. Remove protective foil from the back and stick them to the backrest and seat.

Note! Cushions are optional equipment that comes with chair and are not necessary for usage.

Chair is ready to use.

FOLDING

Repeat actions from “UNFOLDING” section, starting from the last.

HEIGHT ADJUSTMENT

To adjust the height of the chair, press the locking pins located at legs, select the desired height and then lock it with a pin. The legs are adjustable in increments of 2.5 cm.

Note: All legs must be set at the same level and stand firmly on the ground.

USAGE

- Lift the seat cushion and remove the wedge. Slide the container under the seat. Make sure that container lid is removed before usage. After use, cover the container with the lid and place the wedge back onto the seat.

Note! Remember to empty the toilet sanitary container after each use.

- When using the chair in shower, remove the sanitary container. Wedge can be removed as well.
- Chair can be used as toilet raiser. Remove sanitary container and wedge, placing chair over the toilet. After use, wipe out seat surface and let it dry.

4. Technical data

Overall width: 54 cm	Weight: 4,83 kg
Overall height: 78-85	Frame colour: white
Seat height from the ground: 43-51 cm	Seat dimensions: 42 x 38 cm
Maximum capacity: 136 kg	

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.

- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the sanitary container and the seat after each use.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

10. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745)		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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