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Description of symbols that have been printed on the label, Master Carton and or User Manual

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1. Introduction

We would like to thank you for choosing the MOBIAK S.A. Electric Lift Chair "ESTIA III". The product User Manual is an important document that provides you information on the technical characteristics, operating modes, and after-sales support of the product.

- **Please read this user manual carefully before using the product This manual contains specifications, functions & assembly methods of the product.**
- **Learn the product and practice before you will use it.**
- This manual applies to our models: **0811955.**
- Please provide this manual for reference when other people are going to use this product.
- The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design. Please check it carefully.
- The product you have in your hands may differ from the one in the following photos. The assembly process is similar.
- Contact with your dealer if there is any ambiguity or question.
- Improper use of any product may lead to injury. Unsafe use could harm yourselves and others.
- This product is designed to transport 1 person only.
- In case of damage or malfunction of your product DOT NOT use the product and please contact with the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- It is always recommended that you follow the procedures designed to deal with emergency situations.
- Always follow your doctor's instructions.

2. Intended Use

The Electrical Lift Chair "ESTIA III" is intended for disabled, sick, and elderly as well as people with mobility difficulties or people with disabilities and limited mobility who need extra assistance for transfer.

It can be used to transport the user to and from bed, chair and/or toilet.

It can be used as a transport chair, as a toilet seat, and as a toilet raised seat.

The Electrical Lift Chair "ESTIA III is suitable for indoor use and the transport speed is ≤ 3 km/h.

3. Product Introduction

3.1. Features and functions of the product

The large opening 180°. Facilitates the transport of the user. This allows you to move the user very comfortably by gently lifting the user from the bed, chair, or sofa.

Electrically Adjustable height. The height of the product can be adjusted with the built-in electric mechanism, so it can be adjusted to any surface without the user get up from it. This also makes it easier for the attendant because it does not require much physical effort.

Safe and easy to use. The wheels with brake, the locking lever and belt on the back, the folding push handles, the removable seat cushion, and the removable commode make the product safe and easy to use.

Improving lifestyle. The ease of transport for the user's daily needs and the convenience of using the toilet, helps to boost the user's psychology.

Comfort. The removable padded seat cushion provides comfort and allows you to sit for longer period of time. Includes a protective cover that is easily removable and washable.

The maximum load is 120kg, suitable for people with various body types.


IP grade IP25. The product has been tested and is rated as IP25.(2: Protection against solid objects larger than 12mm. & 5: Protection against low pressure water jets from any direction.).

Easy storage. The design with the detachable seat part and the foldable base of the product helps to store it easily when you don't need to use it.

3.2. Package Contents

The package contains the main frame x1, the right part of the seat x1, the left part of the seat x1, the front wheels x1, the commode x1, the base for the commode x1, the seat cushion x1, the instructions for use x1, the remote control x1, the charger with the power cable x1 and tool kit x1.

3.3. Details & Structure of the product

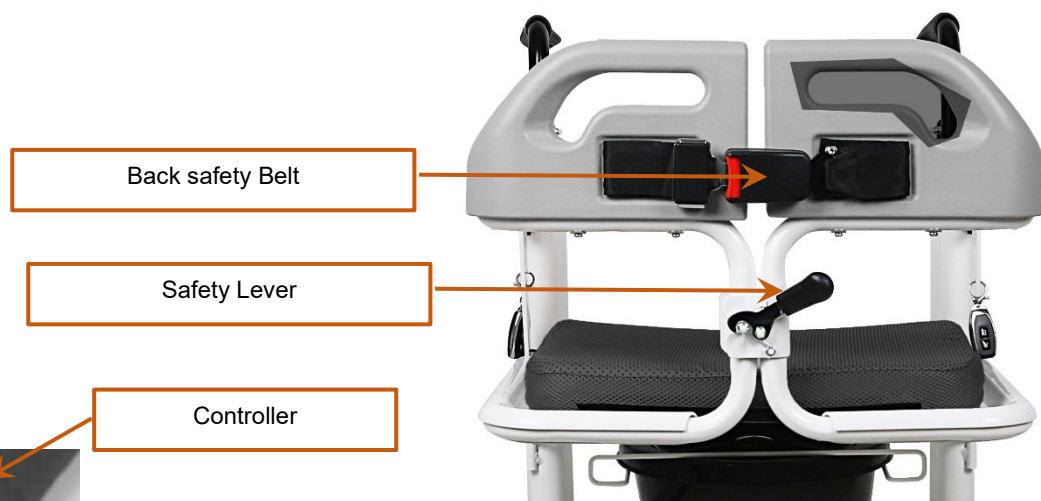- Power switch Button, ON/OFF
- Button for Lifting
- Button for Lowering
- Battery Display

Specifications

Model Reference Number	0811955
Frame	Aluminum
Frame	Foldable & the Seat Section is Detachable
Backrest and Armrest Material	ABS
Seat Material	Steel, welded around with high- strength tubes
Seat Cushion	It has, Removable with washable cover
Rear Wheels	3". With brake & 360° rotation capability
Front Wheels	5", With brake
Brake	On the front & rear wheels
Footrest	Fixed Footplate (Platform)
Minimum radius	77 Cm
With Electric Lifting System	The electric lifting system controls the lifting of the seat base from the vertical part of the frame.
Noise	≤ 65 dB
Battery	Lithium 1 pc X 24V X 4 AH (battery pack is hidden in bottom tube)
Duration Of Battery	After full Charge, with a new battery and 120kg user is estimated for 130 times Lifting & Lowering. Battery life is estimated at 800 charge cycles.
Power Supply	IN: 220V AC, 50Hz, 30W / OUT: 24V-29.4/1A
IP grade	IP25
Max Load	120 Kg
Net weight (Without the seat cushion)	23 Kg

												
0811955	48 cm	46cm	40cm	37 cm	57 cm	73 cm	87-115cm	46-74cm	3"5"	120kg	23kg	

3.4. General safety warnings (read carefully)

- Keep this manual for future reference.
 - If you don't read this user manual, it is preferable not to use this product or another available part.
 - Before using the product, check all components, electrical and non, if they are connected safely and securely.
 - Under no circumstances DO NOT disconnect, cut or modify the parts of the wiring installed or connected to the product.
 - DO NOT use other batteries that are not suitable.
 - Before installation, read the information for battery pack and on battery charger.
 - DO NOT use this product or any available optional equipment unless you have first read and fully understand this user manual.
 - If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
 - Be careful when children are nearby and DO NOT allow children play with the product.
 - Do not exceed the maximum load capacity.
 - Do not smoke & do not use any flame near the product.
 - Special care is always required where there are moving parts that can cause limb entrapment and injury.
 - Do not attempt to lift the wheelchair by any removable parts.
 - If used on sloping ground, all four wheels must be locked.
 - Increase your caution when using the product on carpets, wet or slippery surfaces.
 - The product has been tested and rated with an **IP25** protection level:
 - 2:** Protection against solid objects larger than 12mm
 - 5:** Protection against low-pressure water jets from any direction
- Although the chair offers some waterproof protection, **it is not recommended for full use in the shower**. It can be used in an environment with exposure to water jets, but direct or submerged water exposure should be avoided.
- Never charge the battery while the product is still wet, during or after the shower.
 - Do not touch the power connector of charger with wet hands to avoid the risk of electric shock.
 - Check if the power cable whether is damaged before charging. Do not use damaged chargers and contact your local distributor if this has occurred.
 - Do not use the product on stairs (escalators or not). If there are, do not carry the user with the product. When going up or down stairs, the operator should not move the user along with the product.
 - In case of repair use only original spare parts. Always use MOBIAK approved parts or adapters.

In case of non-Compliance of the above requirements, there may be unexpected injuries or damages.

3.5. Instructions Before Use and Other.

- The Electrical Lift Chair "ESTIA III" should only be used indoors on a flat, non-slippery surface to ensure stability.
- Use should always be under the supervision of a family member, nurse, or caregiver.
- When the user sits or stands up from the product without assistance, it requires the corresponding strength and mobility in the upper part of the body.
- Ensure that it is assembled correctly, the base is fully extended & secured in that position, the seat is on its position securely and the container, if need to be used, is correctly positioned in the appropriate socket.
- The wheels are well fixed.
- There are no cracks or breaks in the frame.
- When adjusting the seat height, the front wheel must be locked. The operator uses the electric switch to control the height.
- Adjust to the appropriate height and ensure that the seat frame on both sides remains at the same level.
- When the user sits or stands up from the product, make sure the wheels are locked.
- When the user is sitting or standing up from the product, the user should not stand on the frame as there is a high risk of tip over. The user must stand on the ground during this procedure.
- When using the Lift Chair, with the user on, and before you start the transfer, make sure that the back safety lever and the back safety belt are locked.
- While using the chair, do not pick up objects on the ground.
- When sitting or standing up, you must push down on the handles so that the wheeled lifting chair remains stationary and does not slip.
- When sitting or standing up, vertical force must be applied to the handles so that the Lift Chair with wheels remains stationary and does not slip.
- When standing up, you must use both handles equally. If you are using only one hand (and without a companion) can cause damage or injury.

WARNING :

When using the electrical lift wheelchair, in the shower, before entering the area, always turn off the central ON/OFF switch and always cover the charging socket with the special cover.



In case of non-Compliance of the above requirements, there may be unexpected injuries or damages.

4. Assembly, charging and handling Instructions.


4.1. Assembly Instructions.

1. Take the main frame of the product out of the carton, pull the two $\Phi 8$ "D" pins from the corresponding sides of the base, then from both sides pull down the legs of the base in the appropriate position, reinsert the pins to stabilize it.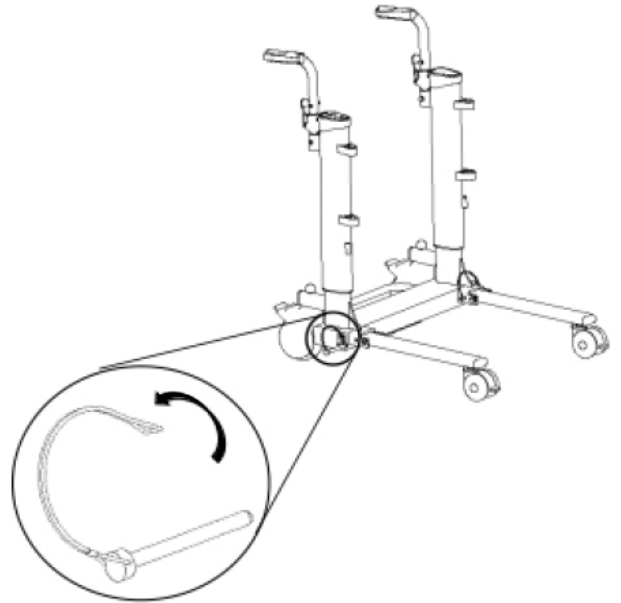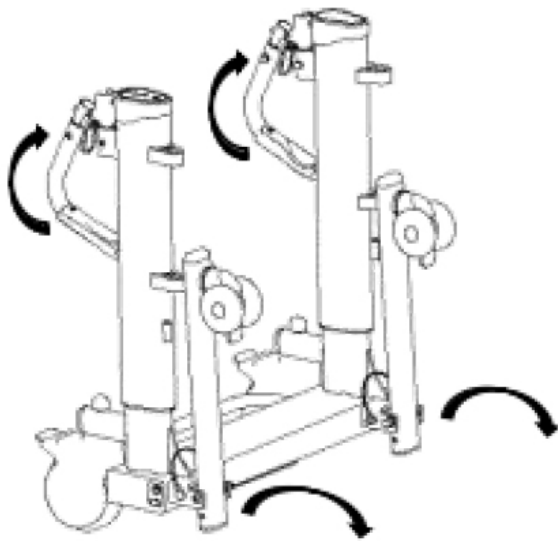



2. Move the left and right handles up to the use position.

3. Place the right seat section directly over the slots on the right frame tube. the holes are aligned, so you need to align the part as well. Press down and hold the quick release pin on the right section until it snaps into place. (use the same method to the other part of the seat)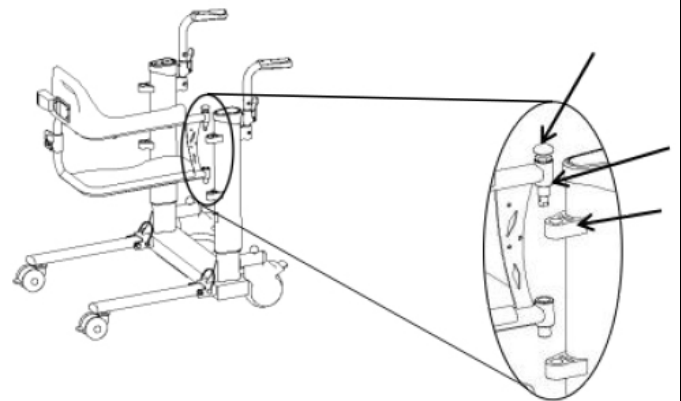

4. After installation and debugging, the seat frame is not obviously loose and can be rotate flexible.


5. If need to use the commode, first install the commode support into the mounting holes of the two uprights, and then install the commode from the rear.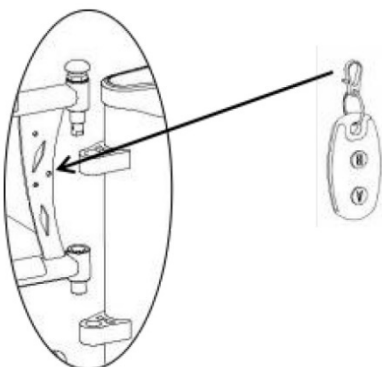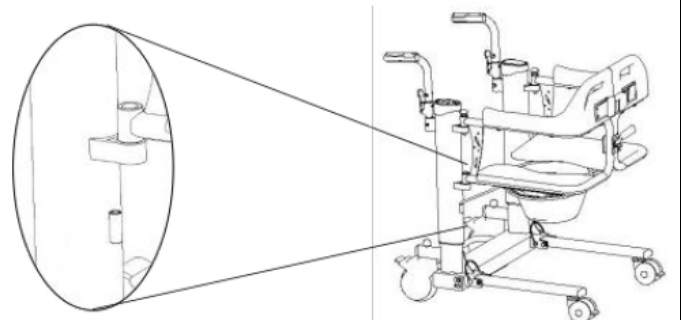

6. The wireless remote control is hung on the side plate holes of the left or right seat brackets.

4.2. Charging Instructions.

1. You must use the charger provided in the package to ensure safe use.
2. Open the waterproof cover of the charging base and first insert the charger plug into the socket on the product and then the power cord, which is at the other end of the charger, into the power plug.
3. During the charging process, the indicator light on the charger is orange, and when the indicator light turns green, it means that the battery is fully charged. When fully charged, unplug the charger from the power plug and the charging base and **close the waterproof cover**.
4. Store the charger in a safe place, for the future use. Turn off the product when not in use to save energy
5. The charging time depends on the remaining power in the original battery, and it is not allowed to exceed 12 hours. FOR FIRST USE the battery needs to be charged for 4-6 hours so that it will be fully charged.
6. When charging the product, and to avoid danger, the product must be turned off (the power switch indicator light is off).
7. Battery life is estimated at 800 charging cycles. When the battery no longer works, purchase a new battery, and replace it as needed or contact the manufacturer for a replacement. Dispose of the old battery at a qualified recycling facility.

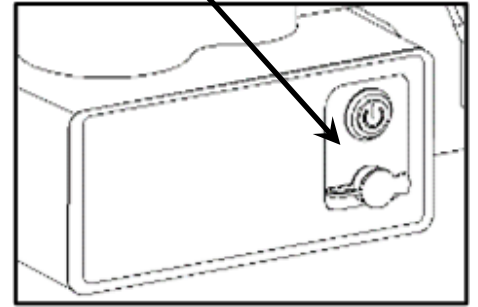4.3. Control system and Remote-control Instructions

1. For the control system, the functions are:


Power switch Button, ON/OFF

Button for Lifting

Button for Lowering

Battery Display

2. For the controller: Button A is the button for Lifting and Button B is the button for lowering.
3. The connection between the Controller and the remote-control will be automatically.

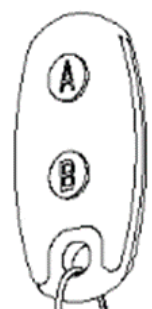Battery Indicator	Light Indicator	Note
Full (3 lights)	Green, Yellow, Red, all lights on	
2 lights indicator	Yellow & Red. Green is off	
1 light indicator	Red only. Green and Yellow is off.	Charging is recommended
Do not use	Red flashes you and hear a beep.	Forced charging.

Note: if the charger is inserted in the ON state, the red, yellow, and green lights will light up sequentially. When all three lights are on, charging is complete.

4.4. Handling Instructions.

1. Let the user sit with a straight back, turn on the power switch (switch indicator light on), the operator presses the lift button or the remote control to adjust the height of the chair. Adjust the high of the chair so the lower surface of the seat is slightly higher than the level of the user.
2. Open the seat section on both sides, and move it in front of the user, adjust the lift chair so that the center of gravity of the user's body is in the center of the seat frame, and adjust the seat height again as so the bottom of the seat frame slightly touches the level of the user's seat.
3. Lock the front wheel brake, lift the user's foot and place it on the footrest. Let the user turn his body slightly to the left and right, and at the same time, the operator rotates the corresponding side of seat frame (left or right), allowing the user to sit on the left and right seat part of the product.
4. When the user is fully seated on The Electrical Lift Chair "ESTIA III" seat, and the left and right parts of the seat are fully closed, the operator must lock the locking lever and seat belt on the back.
5. Then the user can adjust his/her posture so that he/she will be comfortable and place his/her hands on the armrests.
6. The operator checks the distance between the bottom surface of its seat and the surface where the user sat before, raises the frame by 2-3 cm, releases the front wheel brake and slowly moves the chair away from the spot using the push handles.
7. The operator uses the push handles to take the user to the desired place.
8. Follow the reverse procedure to transfer the user from the product to a chair, bed, or couch.

5. Storage and Transport Conditions.

- The product should be stored and transported packaged under the following conditions.
- Temperature range: -20°C ~ 60°C.
- Relative humidity range: 25% ~ 95%.
- Atmospheric pressure range should be 86~106 kPa.
- For storage, the product must be placed in a dry and ventilated place and must not be placed in an environment with high temperature and rapid temperature changes. the product must be isolated from chemicals and corrosive substances.
- For transporting, handle with care.
- When packed, the maximum stack height is three layers.
- DO not store the product for a long term into places which get close to high temperature source or direct sunlight.
- If you need to store it for a long time, fist fully charge the battery and then turn off the power switch.
- If the product has been stored for more than a month, fully charge the battery, and then use it or continue storage. That is to check the level and health of the battery every month.
- After long-term storage, all batteries in the first charge cycle provide less capacity than rated capacity. Full capacity can be reached after several charge/discharge cycles, usually at least 3 cycles.

6. Maintenance

The purpose of periodic check of the product, is to confirm its proper operation as an aid and must be done at least once a month.

- Check the container status for any breaks, cracks, or leaks.
- Check the frame screws. If any are found to be loose, please tighten them.
- Check the plastic parts and frame of the product for any looseness, cracks or breaks.
- Check the stability of the product.
- Check the wheel brakes and confirm their operation.
- Check the condition of the front and rear wheels.
- In case of repair, use only authentic spare parts.

WARNING

In the event of a problem with any of the above, **DO NOT** use the product until the issue will be solved.

11. Cleaning

- The user or the attendant of the wheelchair is responsible for the cleaning, inspection and maintenance.
- In addition to maintenance and inspection, a general cleaning for the frame and the upholstery can be carried out.
- DO NOT leave the product wet or dirty with dust or mud.
- The frame should be cleaned regularly. We recommend cleaning the frame and seat with a mild detergent or antiseptic solution. For your own convenience you can use a spray bottle after adding the solution. Spray the product and then clean with a sponge. Wipe the surfaces dry with a clean cloth.
- Use a clean and soft cloth to wipe the surface of the product.
- The commode must be cleaned immediately after use. First, rinse with plenty of warm water. Then use a soft cloth soaked in a mild detergent or antiseptic solution. Wipe with a clean, soft cloth.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.
- CAUTION: Do not use acidic or caustic solutions.
- The pillow cover is removable and hand-washable in cold water.

8. Environmental Protection Statement:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

9. Incident Reporting:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

10. Declaration Of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

11. Terms of Warranty:

The product is guaranteed for a period of two (2) years warranty on the frame and six (6) months the battery, from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, wheel tire, safety belt, footrests or brakes.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

When the user or the provider apply on warranty request, provide a valid proof of purchase and the completed warranty form, with a photo of the product serial number and a photo or video of the problem that has occurred. In the event the user cannot provide the above documents, if we can confirm that the product or part is still within the warranty period, we may be able to honor the user's request for coverage under warranty, but we are not obligated to do so.

Warranty Form

BUYERS DETAILS:	
FULL NAME:	
PHONE:	
PURCHASING DATE:	
INVOICE NUMBER:	
SERIAL NUMBER:	
LOT:	
DEALER DETAILS:	
FULL NAME:	
ADDRESS:	
PHONE:	
DEALER SIGN AND SIGNATURE	

MOBIAC S.A
KATHIANA AKPQTHPIOY, XANIA - KPTH
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

12. Disclaimer:

The content of the product manual is the intellectual property of MOBIAC S.A. (hereinafter referred to as the "Company") and the copyright is reserved by the Company. Without the written consent of the Company, no one may copy, export or translate the manual or any content included in any form into other languages without the written consent of the Company. The Company carefully compiles the product manual with a sense of responsibility for users but does not guarantee the completeness or correctness of its contents. The product manual, as a purely technical document, does not include implied content or content with oblique reference to third parties. The Company assumes no responsibility for any typographical errors that may cause ambiguities. In the event of direct or indirect loss of information or business interruption due to the product manual or the product information contained therein, the Company shall not be liable.

When using all MOBIAC S.A. products, users must strictly follow the product manual and the instructions for use available on the Company's official website (www.mobiakcare.com) for the handling of the equipment and not violate any warning marked in the product manual when using the equipment. In case of injury to users or other persons, or material damage caused by the handling of the equipment without following the instructions for use or by violating the warnings marked in the product manual, the persons concerned shall bear the respective legal responsibilities.

As the Company continues to improve and update its products, it is essential that you regularly log on to its official website to refer to the most up-to-date instructions and conditions of use or contact your distributor for the latest user manual. The Company reserves the right to revise the product manual without notice.

The Company reserves the right of final interpretation.

 MOBIAC S.A.
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

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