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INTENDED USE:

These products are actively used in cases where support to the abdominal area is needed such as following operations in the abdominal area and after giving birth. Made from a soft material that fully adapts to the body. Comfortable and lightweight, they apply slight compression that is useful for patients with mild pain or seeking long-term treatment.

INDICATIONS:

Mild or moderate back pain, mild post-traumatic injuries, prophylaxis in sports activities and muscle weakness, sciatica, humeral spondyloarthropathy, muscle-ligament injuries, lumbar intervertebral disc herniation.

CONTRAINDICATIONS:

In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.

If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.

Do not use in case of skin allergies due to contact with biocompatible materials.

Discontinue use if area begins to redden and/or excessive heat builds up is occurring.

CAUTION (!):

- The use of the product is indicated after a medical opinion and the duration and monitoring of the treatment is determined by the attending physician.
- Read these instructions carefully before using the product. If you have any questions, consult your doctor or the store where you bought it
- The general instructions for use and the specific indications determined by the doctor must always be followed.
- Treatment must be frequently monitored by a specialist. If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately.
- Although the product is not for single use, it should only be used by one patient. The product must not be reused by other patients.
- Use of this product for purposes other than those specified in this manual is prohibited.
- MOBIAK S.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A. reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

GENERAL SAFETY WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Its application is the sole responsibility of the treating physician.
- The Customization of the product must be done by qualified health personnel.
- The frequency & duration of use should be determined by the treating physician.
- For maximum effectiveness you must respect the daily usage duration according to the instructions of the treating physician or specialized health personnel.
- To ensure the full therapeutic effect, you must use the correct size.
- Do not wear the product while having a bath or a shower.
- For the product to function properly, must be regularly checked to ensure that it is in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.

BEFORE EVERY USE:

- In case of damage do not use the product and contact your dealer for further instructions.
- Always check the condition of wear for the parts to ensure the absolute safe use.
- Regularly check that the straps are tense and in good condition (free of grease, cuts, tears, etc.) and that the velcro fasteners fasten well.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

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WARRANTY:

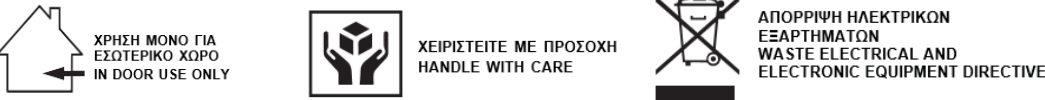
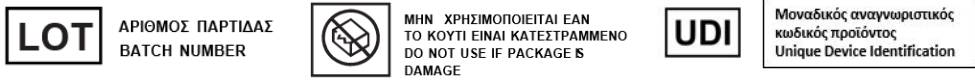
The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by the dealer.

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**MAINTENANCE, CLEANING & DISINFECTION INFORMATION:**

To guarantee a safe use and adequate standard of hygiene, the user should perform these cleaning procedures.

1. Clean the product frequently.
2. Remove the evident dirt before proceeding with the most accurate cleaning.
3. Close the velcro fasteners before starting the washing procedure.
4. Hand washing at 30°C.
5. Do not use bleach.
6. Do not iron.
7. Do not dry clean.
8. Do not tumble dry.
9. DO NOT leave the product wet.
10. If you need to disinfect the product, use a common, mild disinfectant.
11. It is recommended, to carefully cleaning the Velcro fasteners to remove threads, lint, etc., and thereby preserve correct adhesive properties.
12. After washing, make sure that the product is completely dry before use.

APPLICATION INSTRUCTIONS:

The product is ready for use.

It's fitting and application is the sole responsibility of the treating physician and must be done by him or by specialized health personnel.

Place the belt around the body so that the panels are in the center of the back and the spine is in the middle of it. The underside of the belt should be at pelvis level. The velcro should be in the front area and with enough pressure to properly support the area.

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USER MANUAL: : ELASTIC 3D KNITTED LUMBAR SUPPORT WITH STAYS

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